CHARMANT

【Lens Mounting and Disassembly Instructions】

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
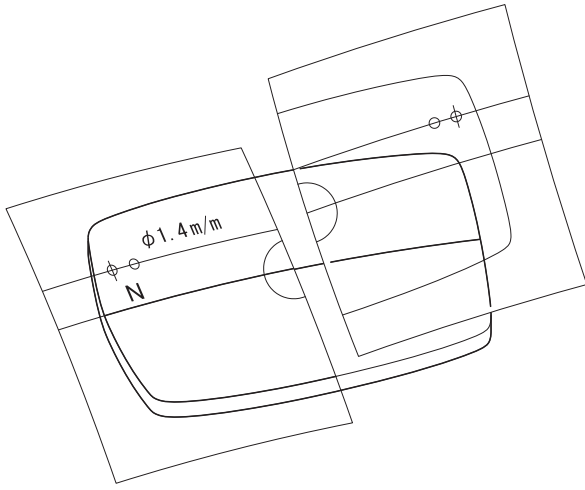
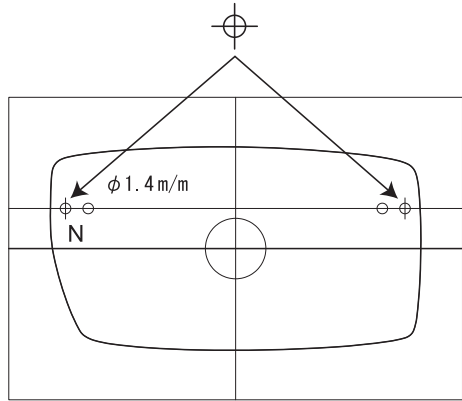
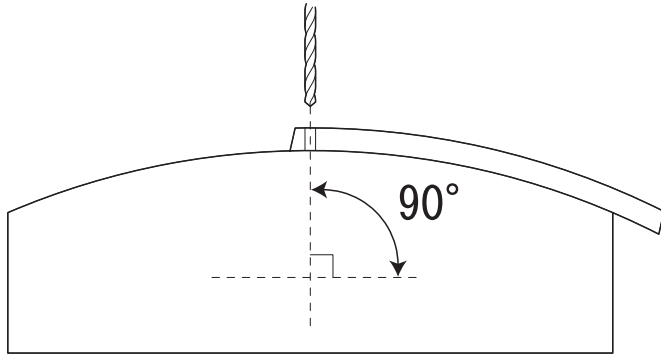
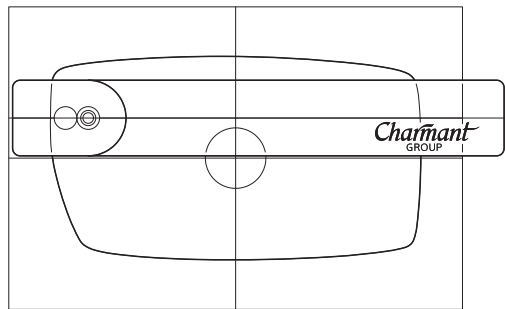
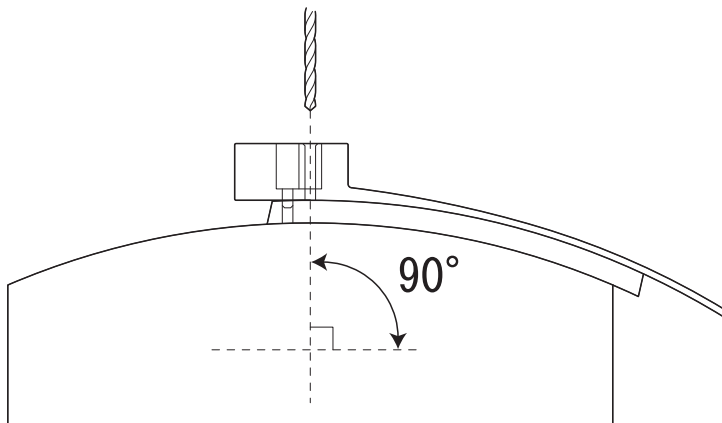
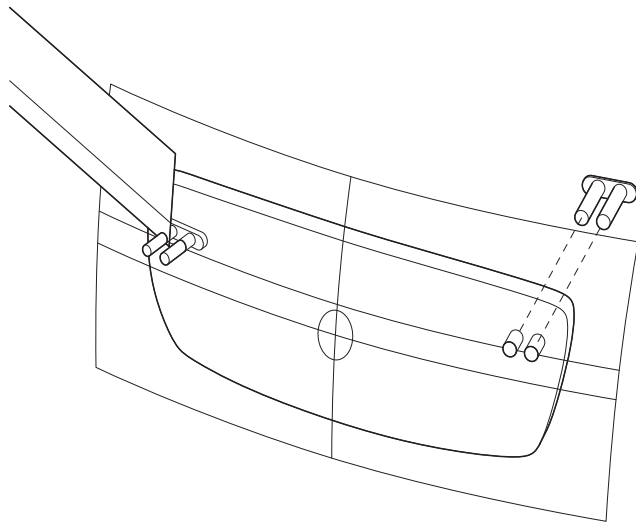
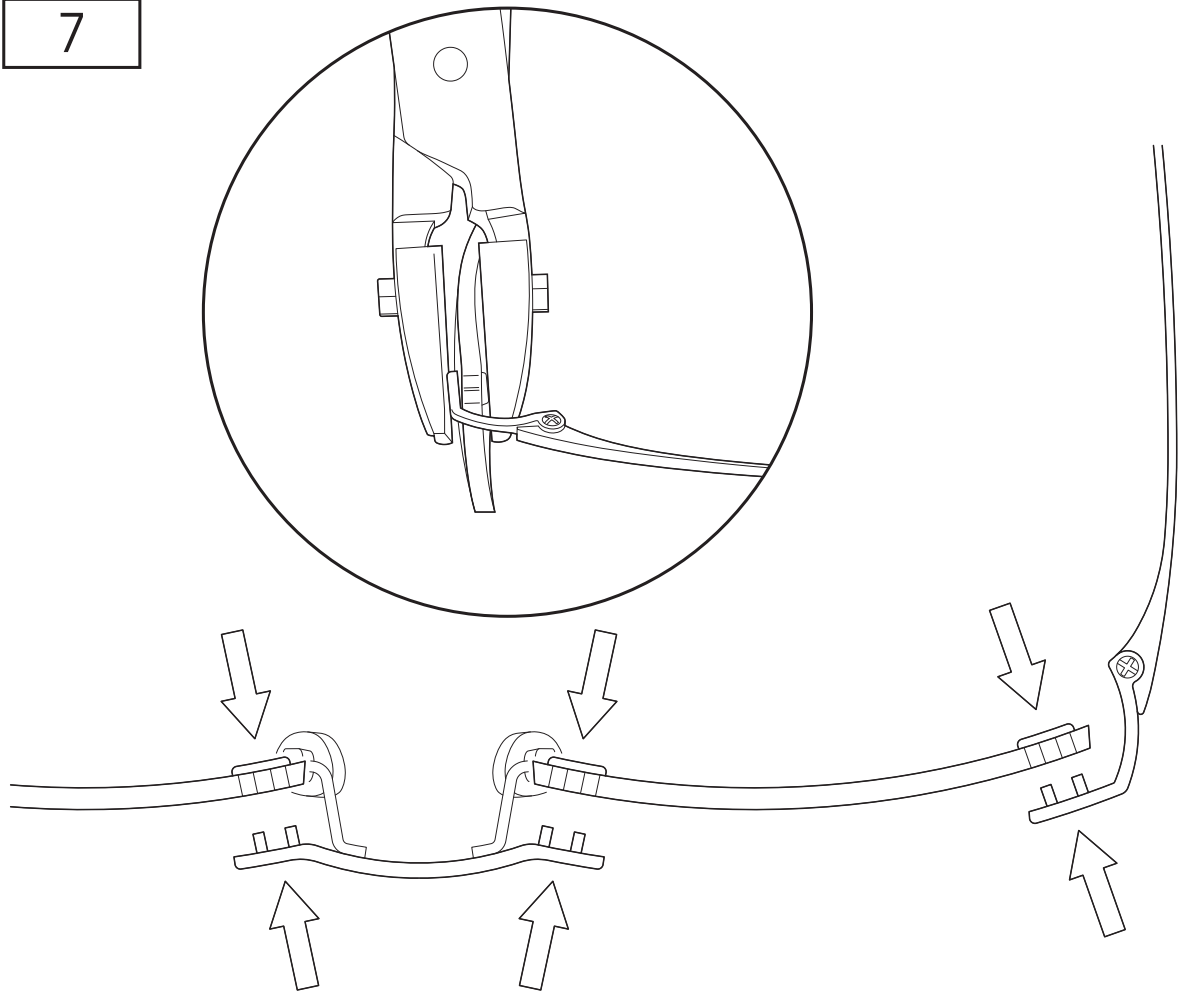
8

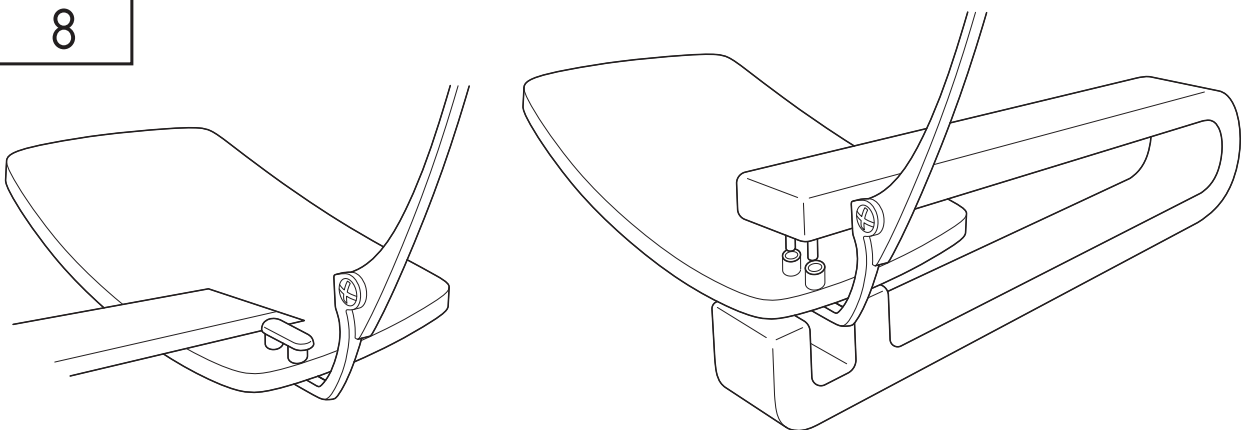【Lens Mounting and Disassembly Instructions】

* A lens edge thickness between 2.3mm to 6.0mm is recommended for this product.

1 . Lens Cutting

Cut the lens by using the lens pattern as per standard procedure for conventional rimless frames.

2 . Making the First Hole

- ① Align the lens sticker carefully with the edge and centerline of the lens.
- ② Check and make sure that the markings for the drillings on the sticker are correctly positioned on both lenses.
- ③ Drill the first hole in accordance with the \varnothing markings on the sticker.
 - * Use a 1.4mm drill bit.
 - * Drill at vertical angle to the plus side of the lens.

3 . Setting the Plastic Gauge

Fix the pin on the underside of plastic gauge into the first hole using the double-sided adhesive tape.

- * Make sure that the horizontal line of plastic gauge is placed exactly on the one marked on the drilling position of lens sticker.

4 . Making the Second Hole

- ① Drill the second hole in accordance with the positioning provided by the plastic gauge.
 - * Use the same 1.4mm drill bit as in the first drilling.
- ② Remove the plastic gauge after finishing the drilling.
 - * Be careful not to peel off the lens sticker when removing the plastic gauge.

5 . Follow steps 1-4 for the opposite lens.

6 . Fixing the plastic parts

- ① Insert the plastic sleeves into the drilled holes from the backside of lens.
 - * Make sure that the heads of plastic parts are flush with the lens surface of minus side.
- ② Hold the head of plastic parts by hand and cut off the plastic sleeves by cutter to be flush with the plus side of lens surface.
- ③ Remove the lens sticker.

7 . Assembly

- ① Insert the metal parts into the holes from the plus side of lens with holding the plastic parts from the minus side.
 - * At this stage, the metal parts don't have to be fixed perfectly.
- ② Using the special resin-coated pliers, press the metal parts by their pins into the holes.
 - * The heads of plastic parts should be flush with the minus side of lens surface.
- ③ Finally, check and make sure that the metal parts are fixed in the hole firmly.

8 . Disassembly of the Demo Lenses / Ophthalmic Lenses

- ① Cut the heads of plastic sleeves at the minus side of the lenses.
 - * For the ophthalmic lenses, be careful not to scratch the surface of the lens.
- ② Separate the metal parts from the lenses by using the special disassembly pliers.
 - * Put and press the metal pins of disassembly pliers on the plastic parts at the minus side of the lenses. Push out the metal parts from the holes.
- ③ Carefully clean the pins of metal parts from any possible residues, making sure to avoid damage of the lacquer surface of the metal parts.

After Care

There is no concern of quality problems in ordinary use, but plastic parts' deterioration may occur depending on the use environment of frame. It is recommended to request wearers to return to their optician once a year for review.

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